

LinkSwitch for Independence!


People with limited physical abilities – those who cannot press the "Play/Stop" button on the Digital Book Player – can now have more independence for reading. Connected through a USB adaptor, the *LinkSwitch* allows users to set a length of time for the book to play, and then use their own accessibility switch to start reading again.

Quiet time between listening times...

For example, in the classroom the teacher chooses the length of time for listening. The student activates his or her own switch to start the reading. When the reading stops, the student has time to process what he or she heard before continuing.

LinkSwitch & Digital Book Player Instructions:

- 1) Set Dip switches on the Swifty (Off On On Off).
- 2) Connect the Swifty to digital book player through the USB port. For a low profile, use a right angle adapter such as the one at www.fasttech.com. Search for item # 1022100, "USB 3.0 Male to Female Vertical Right Angled Adapter." There are similar adapters available from other suppliers.
- 3) On *LinkSwitch* blue side:
 - a. Bottom knob: set to "Media Mode", choose seconds or minutes.
 - b. Top knob: choose how many seconds or minutes to listen.
- 4) On Digital Book Player:
 - a. Press "Power", listen through the intro.
 - b. Press "Play/Stop" button to stop playback.
 - c. To start "Media Mode" listening, press the switch plugged into the LinkSwitch.

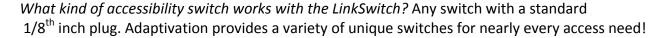



2225 W 50th Street, Ste. #100 Sioux Falls, SD 57105 <u>info@adaptivation.com</u> http://www.adaptivation.com

